

QUICK GUIDE: TRIAL STIMULATOR



1. Stimulation Increase Button
2. Low Battery Indicator Light
3. Stimulation ON Indicator Light
4. Stimulation Decrease Button
5. Stimulation OFF Button





DEFAULT MODE OF THE TRIAL STIMULATOR

- In the default mode, the plus and minus buttons are locked. This locking feature prevents accidental pressing of the buttons, which can change the settings.
- When it is locked, only the Stimulation OFF Button will respond.
- Even in the locked state, you can still receive therapy.
- **Stimulation settings can be changed with the Remote Control.**

TURNING STIMULATION OFF

1. Press .

- Stimulation ON Indicator Light will turn OFF.
-  only turns stimulation OFF. It does not turn stimulation ON.
-  will turn stimulation OFF, even while the Trial Stimulator's buttons are locked.

TURNING STIMULATION ON

- Refer to the Quick Guide - Remote Control booklet.


CHANGING STIMULATION LEVEL


- Refer to the Quick Guide - Remote Control booklet.




ENSURING CABLE CONNECTION

If one of the cables is disconnected from the Trial Stimulator, it may beep continuously. Reinsert the cable using the following steps:


1. Ensure that stimulation is OFF. Press  to stop the alarm and stimulation.
2. Check the color of the sticker at the Trial Stimulator end of the OR Cable.
3. Plug the OR Cable into the Trial Stimulator's port that matches the color of the OR Cable's sticker. A click will sound once the cable is fully inserted into the Trial Stimulator.

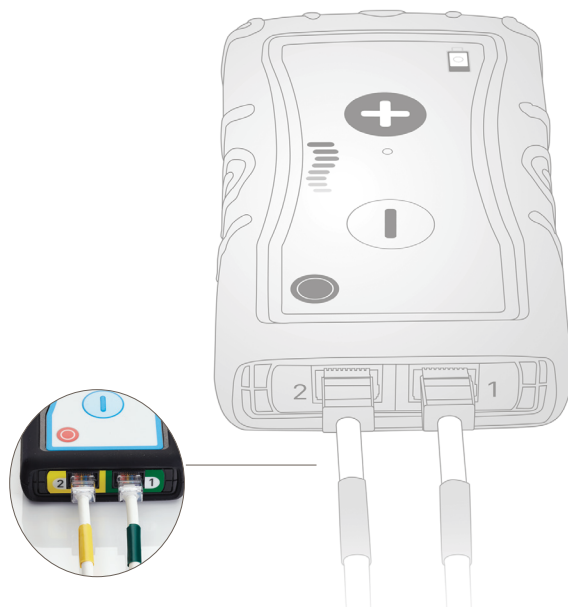

REPLACING THE BATTERIES

When the Low Battery Indicator Light () is flashing or ON, the Trial Stimulator's batteries need to be replaced.

To change the batteries, follow these steps:

1. Turn the Trial Stimulator OFF by pressing .
2. Remove the battery cover on the back of unit and then remove the 2 batteries.
3. Confirm that the new batteries are the purple batteries provided by Nevro.
(These batteries should not be used for any other purpose). *DO NOT* use AA batteries.
4. Insert the new purple batteries into the battery compartment. As with all battery installations, be sure to match (+) and (-) ends of batteries with the corresponding connectors inside battery compartment.
5. Replace the cover and restart stimulation using the Remote Control.

For full instructions on this device, warnings, and precautions, please refer to the Patient Manual.

**FOR MORE INFORMATION
ON HF10 THERAPY,
TALK TO YOUR NEVRO
REPRESENTATIVE.**

All patients do not respond the same way to spinal cord stimulation (SCS) and experiences may vary. Patients should consult a physician to understand the potential benefits and risks of treatment with SCS.



MDSS GMBH
Schiffgraben 41
D-30175 Hannover,
Germany

Australian Sponsor:
Emergo Asia Pacific Pty Ltd
201 Sussex Street, Darling Park,
Tower II, Level 20
Sydney, NSW 2000
Australia



Nevro Corp.

1800 Bridge Parkway
Redwood City, CA USA
www.nevro.com
Tel. 1.650.251.0005
Toll-free 1.888.956.3876

Senza, HF10, Nevro, and the Nevro logo are trademarks of Nevro Corp.
©2015 Nevro Corp. All Rights Reserved.
12124 Rev. B