

PATIENT MRI OVERVIEW



OVERVIEW FOR PATIENTS PREPARING FOR MRI HEAD AND EXTREMITY SCANS

Nevro's Senza® spinal cord stimulation (SCS) system is approved to undergo MRI head and extremity scans under specific conditions. The MRI center will need information from you to make plans for your MRI examination.

WHEN YOUR DOCTOR REQUESTS AN MRI HEAD OR EXTREMITY SCAN:

1. Tell the doctor who requested the MRI head or extremity scan that you have an implanted Nevro Senza spinal cord stimulation system.
2. Ask the following question to your doctor who implanted your Senza system:
 - Can my Senza system safely undergo the ordered MRI head or extremity scan?
3. When scheduling your MRI appointment, provide the following information:
 - You have a Nevro Senza spinal cord stimulation system
 - The contact information of the doctor who implanted your Senza system
 - The Senza MRI guidelines, which can be found at www.nevro.com
4. Bring your patient ID card with you to the MRI appointment
5. Prior to your MRI head or extremity scan, make sure your Senza system is turned OFF. You can turn stimulation OFF with your Remote Control.
6. After the MRI head or extremity scan, turn your Senza system back ON with your Remote Control and contact your pain specialist to ensure that your therapy is programmed to pre-MRI settings.



For additional information, contact your doctor and visit www.nevro.com

All patients do not respond the same way to spinal cord stimulation (SCS) and experiences may vary. Patients should consult a physician to understand the potential benefits and risks of treatment with SCS.

CE
0086 Effective
4 May 2010

USA

EC REP

MDSS GMBH
Schiffgraben 41
D-30175 Hannover,
Germany

Australian Sponsor:
Emergo Asia Pacific Pty Ltd
201 Sussex Street, Darling Park,
Tower II, Level 20
Sydney, NSW 2000
Australia



Nevro Corp.

1800 Bridge Parkway
Redwood City, CA 94065 USA
www.nevro.com
Tel. 1.650.251.0005
Toll-free 1.888.956.3876

Senza, HF10, Nevro, and the Nevro logo are trademarks of Nevro Corp.
©2015 Nevro Corp. All Rights Reserved.
2015032 Rev. B